



# Narcotic Waste via Pyxis

**ALL narcotics should now be wasted in Pyxis. Here's how:**


- Log into pyxis
- Select the 'Waste' button
- Select the appropriate patient
- Select the appropriate med. Note: the initial list ONLY includes meds dispensed in the last 32hrs. If the med isn't included here (ex. Fentanyl patch applied 48 hours ago), select the 'All Meds' button to obtain a list of all meds available in the pyxis station.

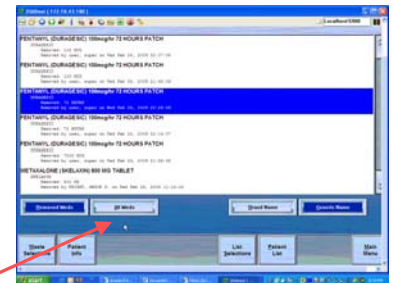

- The first screen says 'Remove Quantity'. Enter the amount **originally removed** from pyxis. [Note the top of the screen still indicates that you are in a waste process.]
- ♦ Then select 'Waste Now'.

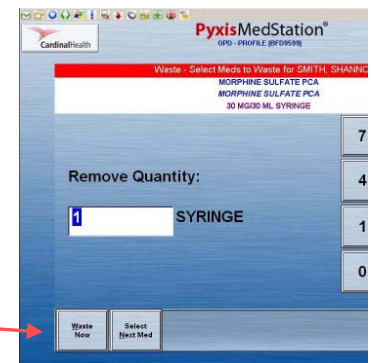

- Enter 'Amount Given'—Pyxis will calculate waste amount.
  - ♦ Fentanyl patch—amount given is the number of hours the patch was in place (ex. 48 hours)
  - ♦ Actiq lollipop—amount given is the approximate amount consumed (ex. 1/2)
  - ♦ PCA syringes should ALSO be wasted via Pyxis.

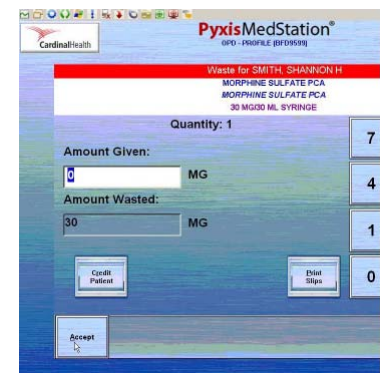

**ALL narcotic waste requires two signatures.**